The MDM Technique of Conscious Sedation: An Advancement in Efficiency and Accuracy


Establishing an effective level of conscious sedation through the inhalation of gases requires: 1. a total flow sufficient to sustain a patient's normal respiratory rate (minute volume), and 2. the appropriate ratio of nitrous oxide/oxygen. A comparison of "the percentage technique" (MDM) to "the increasing nitrous techniques" (all other machines) reveals that both effectively and safely provide conscious sedation. But, the MDM method is more economical, easier to use, and can reduce waste nitrous oxide levels.

Comparison I - TOTAL FLOW REQUIREMENTS:


On all machines with a minimum 3 lpm oxygen flow, the gas mixture is determined by adjusting the flow rate of each gas separately. A patient requiring a 6 liter total volume and a 70/30 ratio, for instance, would receive 7 lpm nitrous oxide and 3 lpm oxygen, resulting in a 10 lpm mixture. The extraneous volume (4 liters) is unavoidable and wasted due to the minimum 3 liter oxygen flow rate.

With the MDM, the gas mixture and gas flow rates are set independently. For a patient requiring a 6 liter volume, gas flow would be set at 6 lpm. The introduction of nitrous oxide would then be dialed in gradually, achieving the 70/30 ratio without affecting the 6 liter total flow.⁽¹⁾

Histogram A-compares the two techniques graphically. Notice the constant flow rate of the MDM technique, regardless of the proportion of gases, while the "increasing nitrous" techniques use increasingly higher total volumes as the percentage of nitrous oxide rises.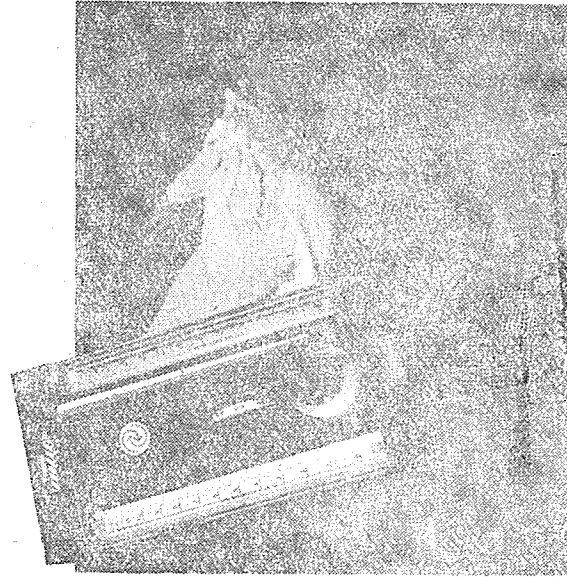

**HISTOGRAM A-COMPARISON:
OTHER MACHINES vs MDM
AT HIGHER MACHINE SETTINGS**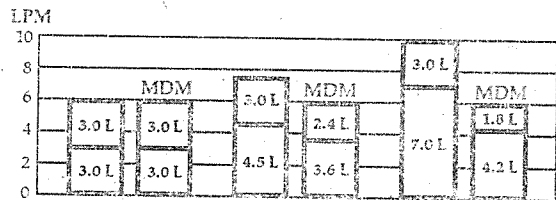


Comparison II - TOTAL NITROUS OXIDE REQUIREMENTS:

When using an open valve on the nasal inhaler to dilute the gases with room air, the volume of nitrous oxide needed is increased, thus resulting in higher gas costs and increased levels of nitrous waste in the operator. (see Histogram B). The MDM technique does not require air dilution.

The implications of air diluting procedures have come into prominence as a result of general concern with waste gas levels in the operator: "Techniques that include the admixture of room air require a higher concentration of nitrous oxide and flow rate from the machine to reach desired but substantially lower alveolar air concentration. This results in both an increase in nitrous oxide air pollution and

increased gas costs. A technique that does not admix room air permits decreased flow rate of nitrous oxide while achieving the same alveolar concentration, thus reducing pollution and cos."²⁾

HISTOGRAMB-COMPARISON: OPENvs CLOSED VALVE ON NASAL INHALER*

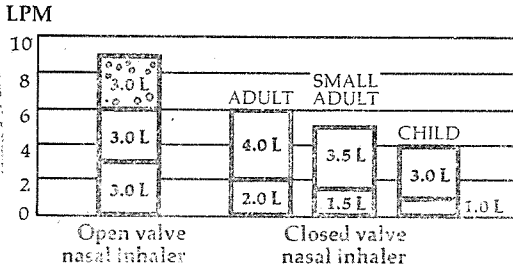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